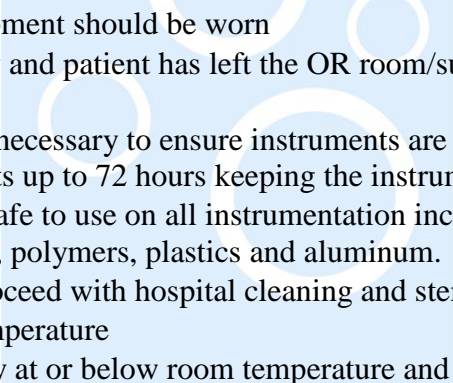


Tri-Power Enzymatic Foam Spray **Instructions for Use**

Tri-Power Enzymatic Foam Spray is specially formulated pre-cleaning foam that instantly becomes active and immediately breaks down bio-burden on contact to prevent caking and encrusting of instruments and scopes prior to cleaning. Contains the same neutral-pH triple enzymatic cleaning action present in the Tri-Power Enzymatic Cleaner. Eliminates messy spills incurred with using liquid pre-cleaners. Ideal for use in operating rooms, endoscopy and angiography suites, dental clinics, cath labs and wherever proper cleaning is immediately required. Foam spray may be used in positive pressure rooms.

- 
- Personal Protective Equipment should be worn
 - Immediately after surgery and patient has left the OR room/suite, spray directly on soiled instruments and scopes.
 - Apply additional foam if necessary to ensure instruments are covered and remain moist. Can remain on instruments up to 72 hours keeping the instruments moist.
 - Clinically validated and safe to use on all instrumentation including scopes, power equipment, stainless steel, polymers, plastics and aluminum.
 - Rinse instruments and proceed with hospital cleaning and sterilization protocol.
 - Store product at room temperature
 - 18 month product stability at or below room temperature and uncontaminated

Note: Follow hospital protocol for handling of cleaning agents. Do not ingest. Avoid contact with eyes. Refer to SDS for product safety recommendations.