

## **Tri-Power Advanced Ultra-Concentrated Enzymatic Cleaner**

### **Instructions for Use**

Tri-Power Advanced Ultra-Concentrated Enzymatic Cleaner is a triple enzymatic cleaning solution formulated to remove fats, proteins, starches and carbohydrate soils. It is a neutral-pH, non-corrosive, low-foaming, completely free-rinsing, all-in-one formulation with detergent included. It's approved for use on all instruments and scopes and its gentle yet effective bio-degradable formulation is safe on gentle instruments, the environment and the end-user alike.



#### **Instructions for Use**

##### **Washer/Sterilizer**

Dilution rate: 1/10 to 1/40oz of solution per gallon of water.

Do not mix with other chemicals or products.

Product works in a broad temperature range (70 - 180 degrees), but optimal washing temperature is 140 degrees.

##### **Ultrasonic Cleaner**

Dilution rate: 1/8 to 1oz of solution per gallon of water.

Rinse thoroughly with water.

May be used in all water temperatures. Optimal temperature is in the range of 70-140 degrees.

Change solution when visibly soiled.