

## **Tri-Power Advanced XP Ultra-Concentrated Enzymatic Cleaner**

### **Instructions for Use**

Tri-Power Advanced XP Ultra-Concentrated Enzymatic Cleaner is a triple enzymatic cleaning solution formulated to remove fats, proteins, starches and carbohydrate soils. It is a neutral-pH, non-corrosive, low-foaming, completely free-rinsing, all-in-one formulation with detergent included. It's approved for use on all instruments and scopes and its gentle yet effective bio-degradable formulation is safe on gentle instruments, the environment and the end-user alike. This product is specially formulated for hard water situations and has a proprietary ingredient that will repasteurize stainless steel. Not only will this make your instruments and washer/disinfector look bright and shiny but will also help remove any staining on instrumentation.



#### **Instructions for Use**

##### **Manual-Wash**

Dilution rate: 1/16 to 1/2oz of solution per gallon of water.

Do not mix with other chemicals or products.

Product can be used in all water temperatures but is recommended to be used in as warm water as tolerable by the person hand washing the instruments.

##### **Ultrasonic**

Dilution rate: 1/16 to 1/2oz of solution per gallon of water.

Rinse thoroughly with water.

May be used in all water temperatures. Optimal temperature is in the range of 70-140 degrees.

Change solution when visible soiled.

Do not mix with other chemicals or products.

##### **Washer/Disinfector**

Dilution rate: 1/10 to 1/40oz of solution per gallon of water.

Do not mix with other chemicals or products.

May be used in all water temperatures, but optimal washing temperature is 140 degrees.