

## **Tri-Power Instrument Lubricant**

### **Instructions for Use**

Tri-Power Instrument Lubricant is a water soluble, neutral-pH lubricant that has been specially designed to not interfere with any type of sterilization method. It is specially formulated to inhibit rust and protects all types of metal instruments and trays, including carbon steel. It forms a protective barrier to keep surgical instruments operating freely and easily.



#### **Instructions for Use**

#### **Washer/Disinfector**

Depending on hospital protocol, set dilution rate in the range of 1/16 – 1 oz. of solution Per gallon of water during lubricant cycle.

Do not mix with other chemicals or products.