

Tri-Power Enzymatic Foam Spray Gel **Instructions for Use**

Tri-Power Enzymatic Foam Spray Gel is specially formulated pre-cleaning foam that instantly becomes active and immediately breaks down bio-burden on contact, and starts the decontamination process to prevent caking and encrusting of instruments and scopes prior to cleaning. Contains the same neutral-pH triple enzymatic cleaning action present in the Tri-Power Enzymatic Cleaner. Eliminates messy spills incurred with using liquid pre-cleaners. Ideal for use in operating rooms, endoscopy and angiography suites, dental clinics, cath labs and wherever proper cleaning is immediately required. Foam spray may be used in positive pressure rooms.

- Recommended to use gloves and face mask.
- Immediately after surgery and patient has left the OR room/suite, spray directly on soiled instruments and scopes.
- Apply additional foam if necessary to ensure instruments are covered and remain moist. Can remain on instruments up to 72 hours keeping the instruments moist.
- Clinically validated and safe to use on all instrumentation including scopes, power equipment, stainless steel, polymers, plastics and aluminum.
- Rinse instruments and proceed with hospital cleaning and sterilization protocol.
- Store product at room temperature
- 18 month product stability at or below room temperature and uncontaminated Product will be effective up to the expiration date as long as the solution remains unsoiled and uncontaminated.

Note: Follow hospital protocol for handling of cleaning agents. Do not ingest. Avoid contact with eyes. Refer to SDS for product safety recommendations.