

**Tri-Power Enzymatic Cleaner**  
**Instructions for Use**  
**ASP100**

Tri-Power Enzymatic Cleaner is a true all-in-one product which has been approved for use in every step of the decontamination process. This includes the sink (hand-washing), ultrasonic, and for all cycles in the washers/disinfectors. It is approved for all endoscopes and safe to use on all Davinci attachments. The only additional product that may be needed, depending on hospital protocol, is an instrument lubricant at the end of the decontamination process. Tri-Power is completely free-rinsing and no residue will adhere to the instruments after use. It is very important that Tri-Power is not mixed with other chemicals/products. Tri-Power Enzymatic Cleaner's manufacturing process is ISO9001:2000 certified and is registered with the FDA as a Class 1 medical device.



**Instructions for Use**

**Manual-wash**

Soak time: 1 to 3 minutes.

Dilution rate: 1/8 to 1oz of solution per gallon of water.

Rinse thoroughly with water.

May be used in all water temperatures but is recommended to be used in as warm water as tolerable by the person hand-washing the instruments. (There is no need to monitor water temperature)

**Ultrasonic Cleaner**

Dilution rate: 1/8 to 1oz of solution per gallon of water.

Rinse thoroughly with water.

May be used in all water temperatures. Optimal temperature is in the range of 70-140 degrees. Change solution when visibly soiled.

**Washer/Disinfector – Standard Concentration**

Dilution rate: 1/8 to 1oz of solution per gallon of water.

May be used in all water temperatures. The optimal temperature is 140 degrees.

Cold water should not be used during the wash cycle.

18 month Product Stability at or below room temperature and uncontaminated up until the expiration date stamped on the container.