

## Tri-Power Rust & Stain Remover

Tri-Power Rust & Stain Remover is specially formulated solution that removes all water deposits, scale and mineral deposits in all washer/disinfectors to ensure optimum performance. Safe to use on all stainless steel or tungsten carbide instruments and restored the luster to instruments and trays. The Tri-Power Rust & Stain Remover manufacturing process is ISO9001:2000 certified.

### Instructions for Use

#### **STAIN REMOVER SOAK:**

- Use 2oz of solution to 1 gallon of water
- Wash and open instruments and soak for 5 to 15 minutes
- Heavy rusting and residue may require light brushing
- When cleaning tungsten carbide soak for 10 minutes only
- Remove instruments from soak solution and rinse thoroughly with water

#### **ULTRASONIC CLEANING:**

- Use ¼ oz per gallon of water and follow ultrasonic manufacture guidelines for cleaning

#### **AUTOCLAVES:**

- Use ½ oz solution per gallon of water
- Run the Acid Rinse cycle for 10 minutes
- Run hot water rinse for 10 minutes