

Tri-Power Advanced Concentrated Neutral Detergent

Instructions for Use

Tri-Power Advanced Concentrated Neutral Detergent is formulated to remove fats, proteins, starches and carbohydrate soils. It is a neutral-pH, non-corrosive, low-foaming, all-in-one formulation. It's approved for use on all instruments and scopes and its gentle yet effective formulation is safe on gentle instruments, the environment and the end-user alike.



Instructions for Use

Washer/Sterilizer

Dilution rate: 1/10 to 1/40oz of solution per gallon of water.

Do not mix with other chemicals or products.

Product works in a broad temperature range (80 - 180 degrees), but optimal washing temperature is 140 degrees.